




CLICK
MEDICAL

QuickFit™ Lacer
by CLICK

OK0240-230-05
Size: Large



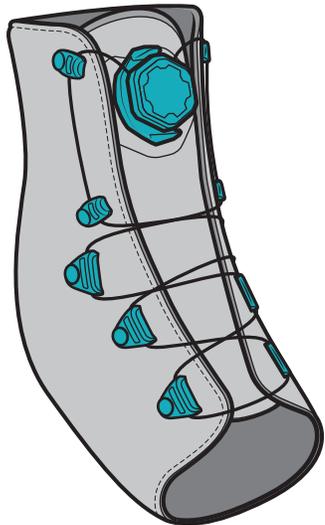

For help and video instructions:
clickmedical.co/quickfit-lacer

Install Hints:

Use QuickFit™ Tool 1/8" punch to make required holes.

If using existing holes for rivet guides, remove metal or plastic grommets from eyelets to ensure a secure rivet setting.

Rivets can be hand snapped to position for verifying fit, but system must be set with QuickFit™ Tool.

Press rivets at 90° with QuickFit™ Tool; pressing an angle will result in poor rivet setting.

Recommended Tools:

- QuickFit™ Setter Tool
- QuickFit™ Rivet Pack
- Marker Pen



Advisory

Must be installed by a trained professional.
Lace will wear. At minimum, inspect monthly.
Overtightening could reduce circulation.
Do not use on patients with nerve issues.
Keep lace away from sharp edges.

See advisories in other languages: clickmedical.co/advisories


 <p>6 MONTHS WARRANTY</p>	<p>WARRANTY US customers contact Click Medical directly: Help@ClickMedical.co Help Line: +1.970.670.7012</p> <p>International customers, please contact your local distributor</p>	<p>MDSS GmbH Schiffgraben 41 30175 Hannover, Germany</p>	
<p>EC REP</p>		<p>Click Medical, LLC 1205 Hilltop Parkway, W101 Steamboat Springs, CO, 80487, USA +1-970-670-7012</p>	

Installation Guidelines:

Dial should be at least 15mm from top edge of device tongue. Lace ports of dial should align with guides.

Top two 20mm guides should sit at the pivot point of the joint, close to the bottom 10mm paired guides, for best results.

Bottom two 20mm guides should be located at distal end.

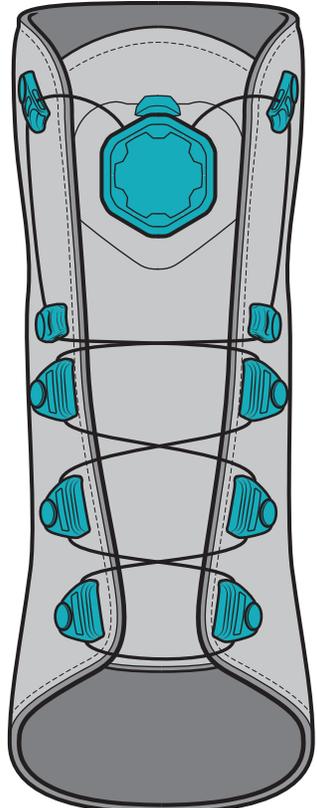


Top two 10mm paired guides should be located near the top of the collar of device.

Bottom two 10mm paired guides should be located above joint.


Middle two 20mm guides should be evenly spaced between top and bottom 20mm guides, for even pressure.

- 1 Determine guide and dial locations using installation guidelines. Punch holes using 1/8" punch option of QuickFit™ Tool.

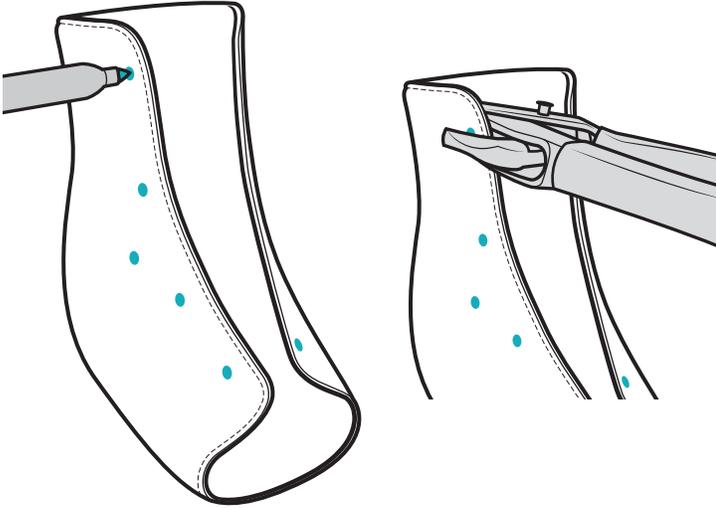


- 2 Use sticker to position system on device:

- a) If tongue is not removable, peel sticker from its backing paper and use it to position lacer on device.
b) If the tongue is removable, keep sticker in contact with its backing paper and hand-set top two guides.

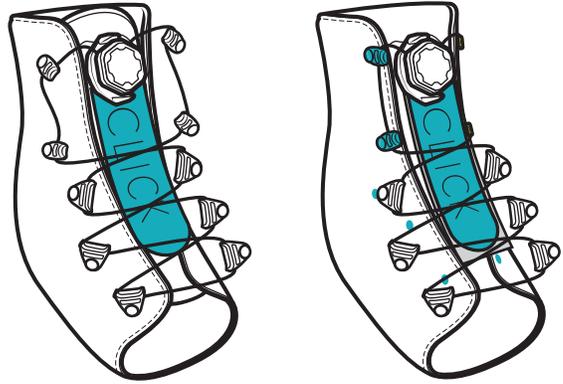


- 3 a) Attach lace guides by inserting male rivet from inside the device and through lace guide.


- b) Snap female rivet cap on top of lace guide, holding the guide to the shoe.

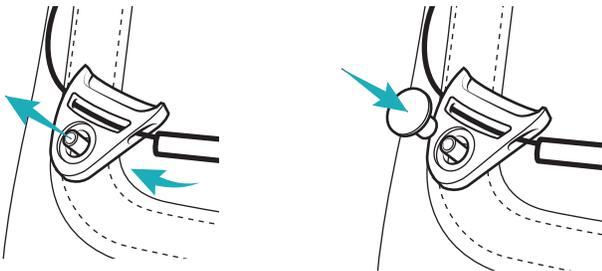

- 4 If satisfied with all placement, secure rivets using QuickFit™ Tool.

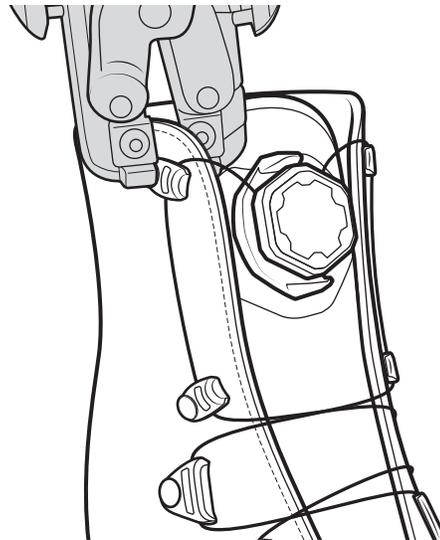

- 5 Rotate guides to correct lace alignment (see dashed line).

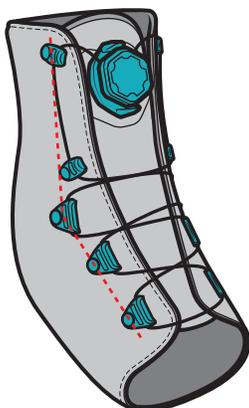

- 6 Cycle system to test function: Push to engage. Twist to tighten. Pull to release.

