
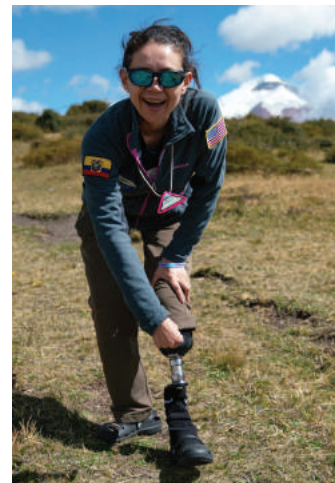


A grayscale photograph of a hand adjusting a prosthetic limb. The hand is positioned over a circular adjustment knob on the limb, which has the word "CLICK" printed on it. The limb itself has a textured, woven appearance. The background is blurred, showing what appears to be a person's leg in a prosthetic. The overall image is dark and moody, with a teal bar at the bottom.

CLICK[®]
■■■■■■■■ MEDICAL

The Guide to Adjustability

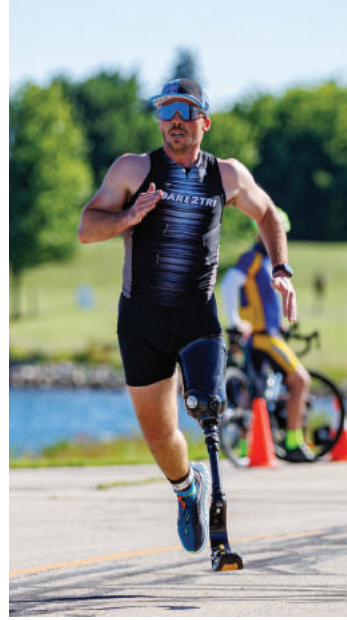

Let's Improve *Lives*

Since 2014, Click Medical's mission is to empower users to overcome limitations, regain mobility and reclaim independence.

Born on the ski slopes.

WHO WE ARE

It all started in 2009, when Certified Prosthetist Joe Mahon was on the ski slopes of Utah. He got a call from a patient who needed him to adjust her device.

He then noticed snowboarders adjusting their boots quickly and easily with the turn of a dial. A light bulb went off. What if that same system could be used to create adjustability in a prosthetic device?

With that idea, RevoFit® was born, creating the first adjustable-socket technology that allows patients to control the fit of their device with a simple turn of a dial.

The rest is history.



WHAT WE DO

Adjustability for *everyone*

Every day we adjust hundreds of things — belts, mirrors, shoes, office chairs, beds, car seats. The list goes on and on. Adjustability isn’t a “nice to have.” We expect it.

So why do patients still have rigid orthotic and prosthetic devices? They put devices on their bodies every day that, to fit and be comfortable, require a complicated system of socks. Or, need you, their provider, to refit their device again, and again, and again.


Click Medical makes the only adjustable solution for custom O&P devices. You can put adjustability in your patient’s hands. **They’ll have the power to micro-adjust their fit with the turn of a dial.**



Patient Kim “Tin Man” Acheson, with Hanger Clinic Provider Leah Bowman, in Eau Claire, WI

WHY WE DO IT

Why Click Medical® Adjustable Solutions?

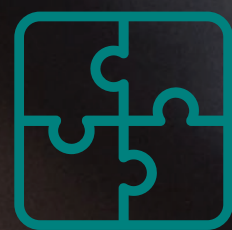

Fit Matters

Adjustability gives you the power to design a device that allows your patients to adjust their fit with a turn of a dial.



Fewer Re-Fits & Re-Makes

When patients can adjust their fit, you will spend less time re-fitting or remaking devices.



Endless Design Options

Be the hero by solving for a wide variety of patient fit issues with unlimited design options.



Lifetime Guarantee

The Click Reel is guaranteed for the life of your patient's device.

NEW Click® Reel

offers reliable, repeatable and predictable adjustability



Powerful

Shifts for speed and strength. Fast lace uptake (1:1) automatically shifts to power mode (7:1) for increased power and precise adjustment.


Use for patients up to 300lbs/135kg.

Bi-Directional

Micro-adjustable when tightening and releasing tension.

Durable

Metal components ensure safety and long life.

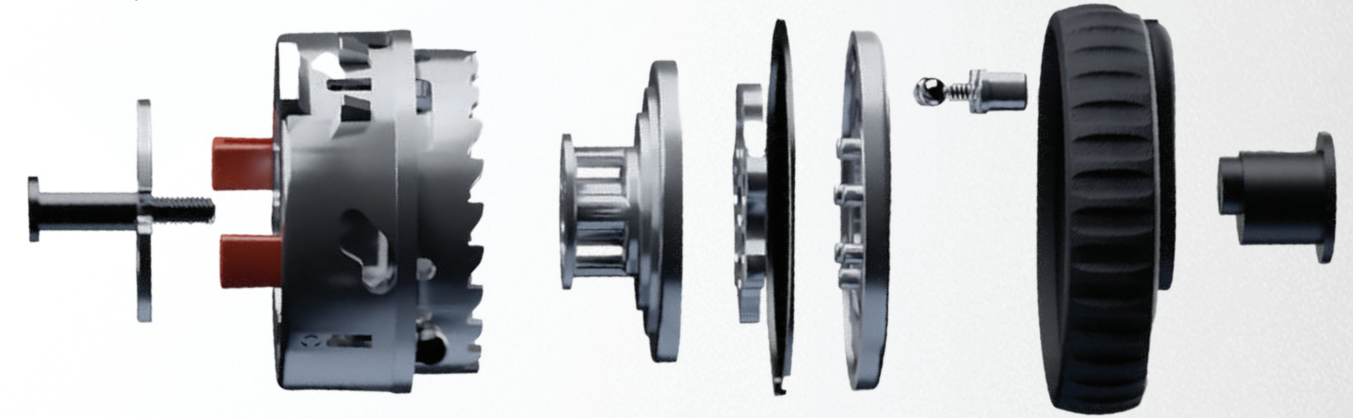

Low Profile

The reel's 24mm streamlined profile seamlessly integrates into devices.



Guaranteed For Life

The Click Reel is guaranteed for the life of your patient's device.

Add adjustability in an instant



Amy Cutler, Laboratory Manager from Agile Orthopedics applying RevoSurface to a device in Denver, CO

Fix the fit *now.*

Retrofit your patient's rigid devices, and give them the freedom to micro-adjust their fit anytime with the turn of a dial.

Prove it *now.*

Experiment and modify devices without refabrication. The RevoSurface Tool Kit parts are easy to remove and reuse.

Print it *now.*

Easily add adjustability to 3D devices.

RevoSurface Solves For:

- ✓ Fit issues on existing rigid sockets
- ✓ The unknowns when verifying design and functionality on check sockets
- ✓ Donning/doffing difficulties
- ✓ Poor-fitting existing devices, especially due to volume changes
- ✓ Lack of flexion and extension control on hinged devices
- ✓ Troublesome Velcro® on cuffs, AFOs, sockets, and shoes

RevoSurface Tool Kit RS-100-07-01

For RevoSurface Tool Kit parts see page 36.



Let your creativity drive solutions.



For detailed videos, tips and tricks enroll in [CLICKACADEMY.co](https://www.clickacademy.co)

NEW RevoFit® Lamination Kit with the Click® Reel

Build adjustability anywhere



Shane Pope during the ROMP Volunteer week in Quito, Ecuador

Do what you do even better.

Endless design options let you solve for each client's unique fit challenges, and improve their overall health and wellbeing.

Save time and money.

With RevoFit, you'll spend less time and money on remakes and refits. Go ahead and take Saturday off!

Drive positive outcomes.

RevoFit adjustability lets patients control their fit and comfort which leads to regular use of their devices, increased activity, and better health.

RevoFit Kits are best for patients with:

- ✓ Significant volume changes
- ✓ Rotation issues
- ✓ Weight and activity changes
- ✓ Distal tibia/femur pressure
- ✓ Limb levering in socket
- ✓ Skin breakdown issues
- ✓ Bulbous distal ends

RevoFit Lamination Kit RF-200-07-01

Available in a Fabricators 10-Pack (RF-200-07-10).
See page 38 for more information.



For detailed videos, tips and tricks enroll in
CLICKACADEMY.co

Certified Prosthetists'

Preferred Adjustable Solution



“Fabricating with RevoFit is a breeze; having adjustability has reduced the number of remakes needed.”


- **Shane Pope,**
CFab Manager, Kenney Orthopedics

“By adding Click Medical solutions to our proven socket designs, we created a superior adjustable device — a win-win for patients and for our practice.

Scott Sabolich, CP, LP
President Ottobock Patient Care – North America



Prosthetic Solutions Guide

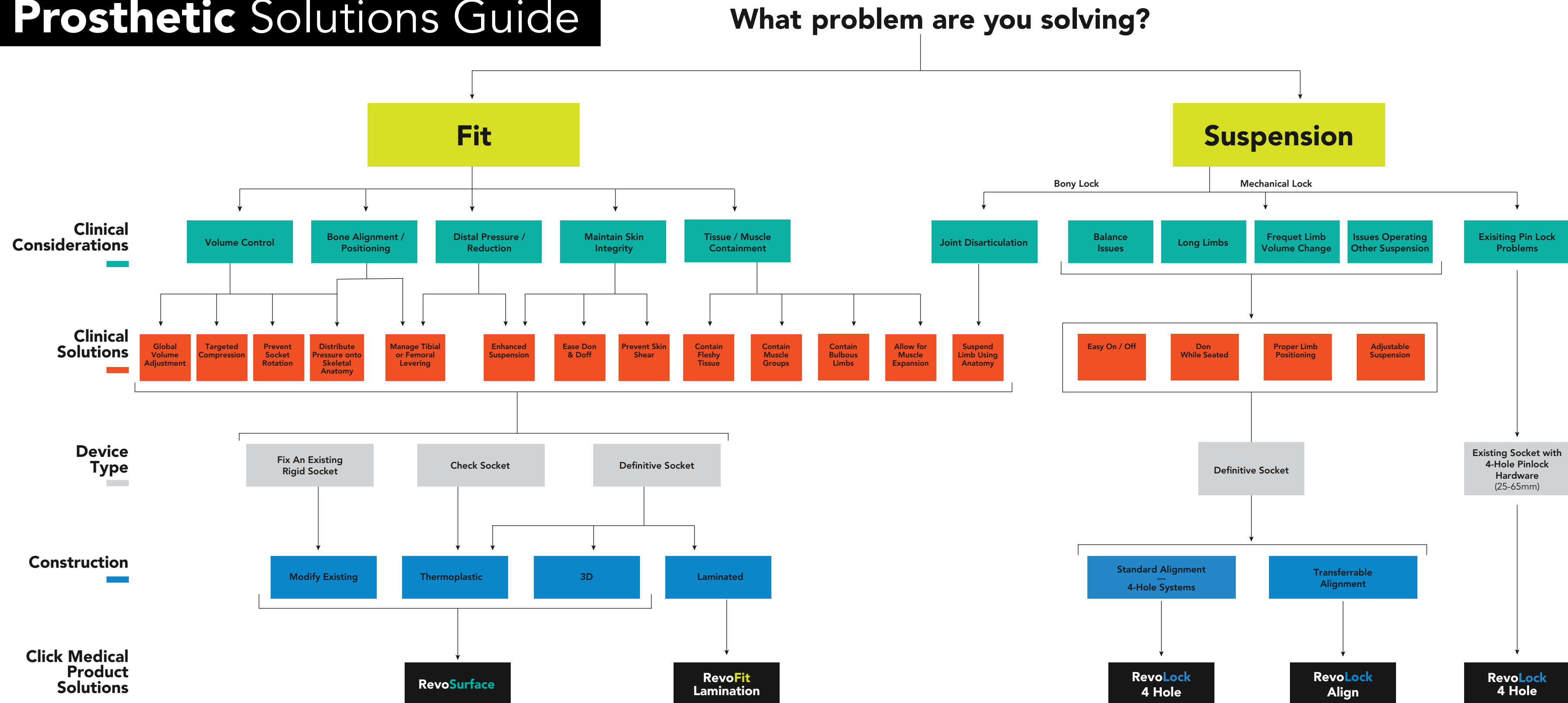

Orthotic Solutions Guide

Clinical
Considerations

Clinical
Solutions

Construction

Click Medical
Product
Solutions

What problem are you solving?

Manage Pressure

Diabetic Charcot Foot Ulcers Fractures Degenerative Diseases

Distribute Pressure Immobilize Unload / Offload

2 Stage Thermoplastic AFO Leather AFO CROW Walker Carbon AFO / KAFO --- Ground Reaction Brace Leather AFO

RevoSurface QuickFit Strap + Buckle QuickFit Lacer RevoFit QuickFit Lacer

Manage Gait

Swing Issues Knee and / or Ankle Weakness Stance Issues Balance

Resist Motion Stop Motion

Solid Thermoplastic AFO Articulated AFO with Dorsiflexion Stop

QuickFit Strap + Buckle RevoSurface QuickFit Strap + Buckle

Manage Alignment

Ankle Weakness Knee Weakness

Varus / Valgus Ankle Correction Genuvalgum / Varum Correction

Leather AFO Plastic AFO's Hinged or Solid KAFO Carbon KAFO

QuickFit Lacer QuickFit Strap QuickFit Strap + Buckle + RevoSurface QuickFit Strap + Buckle RevoSurface

Manage Range of Motion

Contractures Drop Foot Excessive ROM due to Weakness

Restore ROM Limit ROM Restrict ROM

Night Splints 2 Stage AFO: with Dorsiflexion Stop Leather AFO Solid Thermoplastic AFO 2 Stage AFO: Thermoplastic Hinged AFO

RevoSurface QuickFit Lacer QuickFit Strap + Buckle QuickFit Strap + Buckle + RevoSurface

Aid Activities of Daily Living

Compliance Issues Hemiplegia or Single Arm Limited Strength Limited Flexibility

Ease Don / Doff Ease Adjustment

Flexible Materials Carbon or Rigid Materials

QuickFit Strap + Buckle QuickFit Lacer RevoSurface

NEW

RevoLock® Align Kit and 4-Hole Kit with the Click® Reel, and RevoLock® Lanyard V2 with BOA® Dial

Safe, reliable limb alignment & suspension

RevoLock Kits are best for patients with:

- ✓ Long limbs
- ✓ Bulbous distal ends
- ✓ Balance issues
- ✓ Strength issues
- ✓ Large and frequent volume changes
- ✓ Insufficient suspension systems



Joe Mahon, CP, Click Medical Co-Founder and Chief Clinical Officer fabricating with new RevoLock Align Kit

Easy

- The Click Reel quickly and easily pulls the limb into position
- Micro-adjustable when volume or activity levels change
- Quick Connect Snap makes it easy to attach to the liner

Safe

- Don or doff while seated, reducing the chance of falls
- Adjustable suspension reduces pistoning and skin shear
- Decreased distal tension while sitting allows for fluid recovery and volume stabilization

Secure

- Secure suspension regardless of limb position
- Quick Connect Snap locks onto the liner and prevents unintentional detaching
- New slider feature locks limb in the socket but still allows for micro-adjustability

Overcome the challenges of suction and pinlock suspension.



RevoLock Align Kit

RL-301-07-01

- For transferable alignment
- Single or double stage lamination
- 38mm build height
- Small/medium limbs, pediatrics, and upper-extremity sockets
- Ideal for use with 3 and 4 ear connectors
- No tool form required



RevoLock Lanyard V2 with BOA® Dial

RL3000-220

- For transferable alignment
- Single or double stage lamination
- Low 25mm build height
- Upper or lower extremity sockets
- Lightweight system
- No tool form required



RevoLock 4-Hole Kit

RL-300-07-01

- For standard 4-hole alignment
- Single or double stage lamination
- 25mm build height
- Lower-extremity sockets
- Easy to retrofit or replace an existing pinlock system
- RevoLock Tool Form allows for easy fabrication



RevoLock Tool Form

RL-310-00-01 (sold separately)

For more information on RevoLock parts, see page 40.

“

Being able to turn the dial until the user feels seated in the socket gives them a sense of security when they stand and ambulate.

— **Regan Thompson, CPO**
at RISE in Denver, CO



Clinical Considerations

Clinical Solutions

Patient Requirements

Construction

Click Medical Product Solutions

Which RevoLock Kit should I choose for my patient?



RevoLock Lanyard V2 with BOA® Dial

- Patients who need simple suspension with 1:1 mechanical advantage when lower powered donning is appropriate
- Patients with good hand strength and dexterity to attach the threaded insert to the liner
- Patients who have experience and preference for the legacy RevoLock Lanyard system

RevoLock Align Kit with Click® Reel

- Patients needing higher 7:1 donning power
- Patients with small limb diameter
- Patients with transferable alignment

RevoLock 4-Hole Kit with Click® Reel

- Patients needing higher 7:1 donning power
- When retrofitting a pinlock
- Patients with standard alignment

QuickFit® Lacer

with the low-power BOA® dial

A preassembled system that easily rivets onto any shoe, cuff, or AFO.

QuickFit Lacers are best for patients with:

- ✓ Diabetes
- ✓ Reduced dexterity due to stroke
- ✓ Compromised dexterity
- ✓ Balance issues



1 Add Onto New or Existing Orthoses

Either use the kit for fabrication of new cuffs and AFO's, or easily upgrade a pair of shoes.

2 Easy Rivet-on Installation

Using the QuickFit Rivets and QuickFit Tool allows installation in under 5 minutes.

3 One-Handed, Micro-Adjustment

Patients can easily adjust their device, quickly adapting to changes in activity or support needs.

QuickFit Lacers are available in four sizes as well as a releasable version. See page 41 for details.

QuickFit® Straps & Buckles

with the low-power BOA® dial

QuickFit® Strap 1.5" (OK1410-130-05)

QuickFit Straps quickly install on a wide variety of cuffs, orthoses, and prosthetic devices, making adjustments easy and more effective.



Micro-Adjustable

Each "click" of the low power BOA dial moves the strap 1mm for the perfect fit every time.

Fast, Easy and Customizable

The cut-to-size QuickFit Straps are prefabricated, durable, and installs in minutes.

User-Friendly

Adjustability allows for a true and comfortable fit, resulting in increased use of devices.



QuickFit® Buckle 1.5" (OK1415-000-05)

We recommend adding the QuickFit Buckle onto the QuickFit Strap for an easy, secure strap that never wears out.



Magnetic Self-Locating Buckle

Allows the user to easily connect the strap. It's perfect for patients with limited dexterity or for placement in hard-to-reach areas.

Easy One-Handed Use

Never Wears Out

The durable plastic interface eliminates the need to repeatedly hook and unhook Velcro®.



QuickFit Straps & Buckles are best for patients with:

- ✓ Hemiplegia
- ✓ Frequent volume and activity changes
- ✓ Compromised dexterity or strength issues
- ✓ Donning and doffing difficulties

Let's start *adjusting*



David Krupa, CPO and Camila Diaz, CPO at Range of Motion Project (ROMP) HQ in Quito, Ecuador

CLICK**ACADEMY**

Learning how to adjust starts *here.*

Curious how to lace the new Click® Reel?

Wonder if the RevoSurface® Tool Kit can help you quickly solve a current fit challenge?

Want to retrofit an existing shuttle lock with RevoLock® 4-Hole?

Look no further. Answers to your questions are in CLICK**ACADEMY**.

There are four courses for each product:

1

Learn It.

Meet the product, and see different design opportunities and clinical applications.



2

Build It.

Chock full of videos, this course walks you through everything you need to know to add or fabricate our systems into your devices.



3

Optimize It.

It's all about fine-tuning! You'll find various great resources, from optimizing fit to replacing parts.



4


Expand It.

Discover endless design ideas. Submit your own design and add it to our growing Solution Series library!

Click Academy is certified in the
United States and Canada



ENROLL NOW

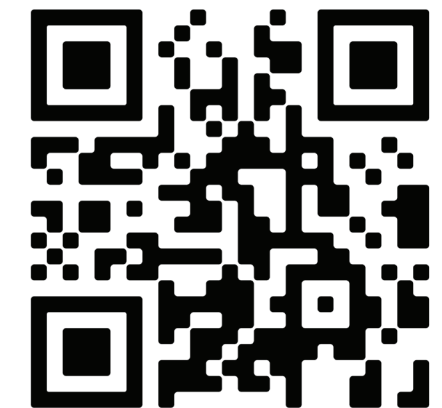

www.clickacademy.co

30+ CEUs available!


“The RevoFit[®] system has enabled Sydney to manage volume changes throughout the day while maintaining secure seal-in suction suspension.

— Scott Moore, CP
Bulow Orthotics & Prosthetic Solutions,
Murfreesboro, TN



Sydney Mashburn,
RevoFit user since 2021



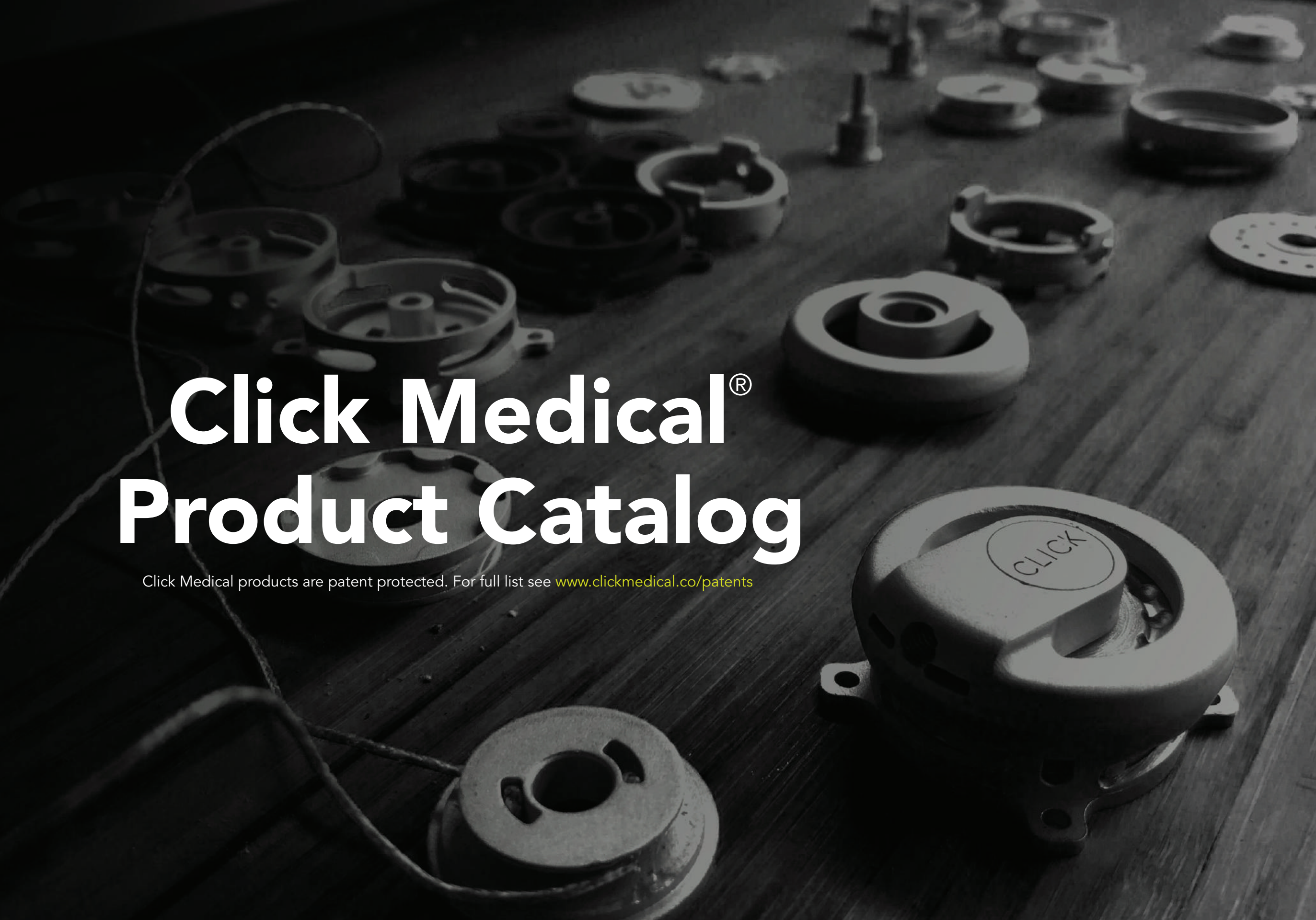

Click Medical® Product Catalog

Click Medical products are patent protected. For full list see www.clickmedical.co/patents

RevoSurface®



RevoFit®



RevoLock®

QuickFit®

Replacement Parts

SKU	Product	Description	Picture
RS-100-07-01	RevoSurface® Tool Kit with Click® Reel	<p>This single RevoSurface Tool Kit with the Click Reel allows you to test adjustable designs and fit of check sockets, 3D printed devices, and on orthoses. It can also be used to solve fit issues on existing rigid devices.</p> <p>The Click Reel innovation creates adjustable devices with a lower streamlined profile, micro/macro adjustment that incrementally tightens and loosens with increased durability.</p> <p>Kit includes: 1x Click Reel, 1x Surface Collar, 1x Surface Shim, 2m HC Lace, 1.8m Lamination Tubing, 1x Metal Lace Feeder, 3x Plastic Lace Feeders, 1x Reel Tool, 10x Surface Guides, 5x Releasing Guides, 4x Phillips Head Screws, 4x Torx Head Screws</p> <p>User Weight Limit: 300lb (135kg)</p>	
RP-001	HC Lace Kit	2m of HC (High Capacity) Lace and Click Reel components for RevoSurface Tool Kit, RevoLock Align, and RevoLock 4-Hole Kits with the Click Reel.	
RP-000-07	Click® Reel and HC Lace Kit	<p>Replacement Click Reel and 2m of HC (High Capacity) Lace for RevoLock Align, RevoLock 4-Hole and RevoSurface Tool Kits only.</p> <p>This is not for replacing a BOA® dial. Please see RP0100-320-05 for replacing a BOA® dial.</p>	
RP-101	Surface Guide 10-Pack	The 10-Pack of Surface Guides allows a practice to have on hand additional guides for easy replacement and configuration changes. For RevoSurface Tool Kits with the Click Reel only.	

SKU	Product	Description	Picture
RP-102	Surface Collar	The Click Reel mounting collar. For surface mount only. Not for use with lamination.	
RP-105	Surface Releasing Tab 5-Pack	5-Pack Surface Releasing Tabs for converting a device from a fixed lace path to a releasing lace path to fully open the device or to replace existing Surface Releasing Tabs.	
RP-201	Lamination Tube - 1.8m	Replacement Lamination Tube (1.8m) for all Revo Kits with the Click Reel and BOA® dials.	
BL-001	HC Lace - 25m Spool	<p>25m spool of HC (High Capacity) Lace assures you will always have replacement lace on hand.</p> <p>Cost effective, sustainable and less waste.</p> <p>Used in the RevoSurface and RevoLock Kits with the Click Reel only.</p>	
AC-100-05	Comfort Grip Ring - Black 5-Pack	Provide a soft-touch feel and 10% increased turning power to the existing Click Reel. Great for patients with dexterity issues.	
AC-101-05	Power Grip Ring - Black 5-Pack	Provide a soft-touch feel and 20% increased turning power to the existing Click Reel. Great for patients with dexterity and hand strength issues.	

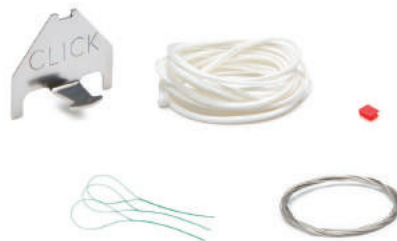



SKU	Product	Description	Picture
RF-200-07-01	RevoFit® Lamination Kit with Click® Reel	<p>This single RevoFit Lamination Kit with the Click Reel allows you to fabricate user-controlled adjustability anywhere on a device.</p> <p>The Click Reel innovation creates adjustable devices with a lower streamlined profile, micro/macro adjustment that incrementally tightens and loosens with increased durability.</p> <p>Kit includes: 1x Click Reel, 1x Lamination Collar, 2m HD Lace, 1x Metal Lace Feeder, 3x Plastic Lace Feeders, 1x Reel Tool, 1.8m Lamination Tubing</p> <p>User Weight Limit: 300lb (135kg)</p>	
RF-200-07-10	RevoFit® Lamination Fabricators 10-Pack with Click® Reel	<p>The RevoFit Lamination Fabricators 10-Pack with the Click Reel is recommended for central fabricators and practices that create large volumes of adjustable devices.</p> <p>Save 10% on purchase price of each single kit while assuring you will always have a kit on hand.</p>	<p>Contents:</p> <ul style="list-style-type: none">○ 10 Click® Reels○ 10 Lamination Collars○ 25m Spool of HD Lace○ 5 Metal Lace Feeders○ 20 Plastic Lace Feeders○ 3 Reel Tools○ 10 Lamination Tubes - 1.8m
RP-002	HD Lace Kit	2m of HD (High Durability) Lace and Click Reel components for RevoFit Lamination Kits with the Click Reel.	
RP-002-07	Click® Reel and HD Lace Kit	<p>Replacement Click Reel and 2m of HD (High Durability) Lace for RevoFit Kits only.</p> <p>Note: This is not for replacing a BOA® dial. Please see RP0100-320-05 for replacing a BOA® dial.</p>	


SKU	Product	Description	Picture
RP-200	Lamination Collar	Replacement Lamination Collar for RevoFit & RevoLock Lamination Kits with the Click Reel only.	
RP-201	Lamination Tube - 1.8m	Replacement Lamination Tube (1.8m) for all Revo Kits with the Click Reel and BOA® dials.	
BL-002	HD Lace - 25m Spool	<p>25m spool of HD (High Durability) Lace assures you will always have replacement lace on hand.</p> <p>Cost effective and sustainable with less waste.</p> <p>Compatible with both the RevoFit Lamination Kit with the Click Reel and RevoFit Lamination Kit with BOA® dial.</p>	
RP-300	Slider Part Kit	Replacement Slider and Lock Wire for dynamic AFOs, large gap prostheses or RevoLock applications.	
RP-309	Slider Lamination Kit	Lamination Kit for fabricating new dynamic AFOs, large gap prostheses or replacement for RevoLock Kits.	
AC-100-05	Comfort Grip Ring - Black 5-Pack	Provide a soft-touch feel and 10% increased turning power to the existing Click Reel. Great for patients with dexterity issues.	
AC-101-05	Power Grip Ring - Black 5-Pack	Provide a soft-touch feel and 20% increased turning power to the existing Click Reel. Great for patients with dexterity and hand strength issues.	




SKU	Product	Description	Picture
RL-301-07-01	RevoLock® Align Kit with Click® Reel	<p>This single RevoLock Align Kit with the Click Reel is for transferable alignment fabrication for both upper and lower extremity sockets.</p> <p>The Click Reel innovation offers a lower streamlined profile, micro/macro adjustment that incrementally tightens and loosens with increased durability. The Slider enhances safety for limbs changing in position or fluctuating in volume.</p> <p>Kit includes: 1x Click Reel, 1x Lamination Collar, 1x Slider Lamination Kit, 2x RevoLock LE Quick Connect Inserts, 1x RevoLock LE Quick Connect Snap, 2m HC Lace, 1x Metal Lace Feeder, 3x Plastic Lace Feeders, 1x Reel Tool, 0.5m Lamination Tubing, 1x Align Housing + Fabrication Post</p> <p>User Weight Limit: 300lb (135kg)</p>	
RL-300-07-01	RevoLock® 4-Hole Kit with Click® Reel	<p>This single RevoLock 4-Hole Kit with the Click Reel is for standard alignment fabrication for lower extremity sockets and is easy to retrofit a standard 25x65mm shuttle lock.</p> <p>The Click Reel innovation offers a lower streamlined profile, micro/macro adjustment that incrementally tightens and loosens with increased durability. The Slider enhances safety for limbs changing in position or fluctuating in volume.</p> <p>Kit includes: 1x Click Reel, 1x Lamination Collar, 1x Slider Lamination Kit, 2x RevoLock LE Quick Connect Inserts, 1x RevoLock LE Quick Connect Snap, 2m HC Lace, 1x Metal Lace Feeder, 3x Plastic Lace Feeders, 1x Reel Tool, 0.5m Lamination Tubing, 1x 4-Hole Puck, 1x Tool Form Plug</p> <p>User Weight Limit: 300lb (135kg)</p>	
RL-310-00-01	RevoLock® Tool Form	<p>The 65mm x 25mm RevoLock Fabrication Tool Form is a required, reusable fabrication tool for RevoLock 4-Hole Kits.</p>	



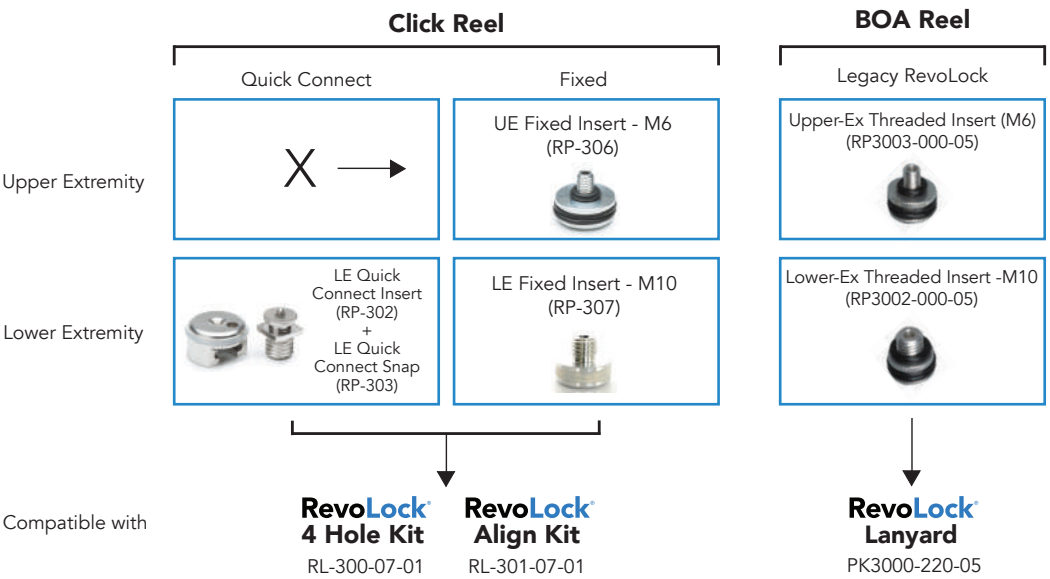

SKU	Product	Description	Picture
RL3000-220	RevoLock® Lanyard V2 with BOA® Dial	<p>This kit is designed to be similar to the original RevoLock Lanyard Kit, including the familiar BOA® dial and the former distal housing/threaded insert. This kit provides a familiar solution that matches what customers are accustomed to when using RevoLock, making it easier for them during our development phase.</p> <p>Kit includes: 1 x BOA® Lanyard V2 Dial, 1 x Lamination Collar, 1 x Lamination Tubing, 1 x Legacy LE Threaded Insert, 1 x Distal Housing 2-Stage Lamination Bolt, 1 x Lamination Post, 2 x 1.22m HC Lace, 1 x Metal Lace Feeder, 1 x Spring</p> <p>User Weight Limit: 220lb (100kg)</p>	
RP-310	RevoLock® V2 BOA® Dial and HC Lace Kit (Port C)	<p>Replacement 1 x BOA® Lanyard V2 Dial, 1.22m of HC Lace (High Capacity), 1 x Metal Lace Feeder. Compatible with sockets fabricated with the Click Lamination Collar. For use with Port C Lace exit.</p>	
RP-310A	RevoLock® V2 BOA® Dial and HC Lace Kit (Port A)	<p>Replacement 1 x BOA® Lanyard V2 Dial, 1.22m of HC Lace (High Capacity), 1 x Metal Lace Feeder. Compatible with sockets fabricated with the Click Lamination Collar. For use with Port A Lace exit.</p>	
RP-312	RevoLock® V2 HC Lace Kit	<p>1.22m of HC Lace (High Capacity), 1 x Metal Lace Feeder. Compatible with sockets fabricated with the Click Lamination Collar.</p>	
RP-001	HC Lace Kit	<p>2m of HC (High Capacity) Lace and Click Reel components for RevoLock Align, RevoLock 4-Hole, and RevoSurface Tool Kits with the Click Reel.</p>	
RP-000-07	Click® Reel and HC Lace Kit	<p>Replacement Click Reel and 2m of HC (High Capacity) Lace for RevoLock Align, RevoLock 4-Hole and RevoSurface Tool Kits only.</p> <p>This is not for replacing a BOA® dial. Please see RP0100-320-05 for replacing a BOA® dial.</p>	




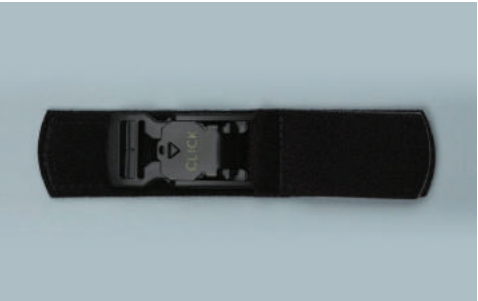
SKU	Product	Description	Picture
RP-200	Lamination Collar	Replacement Lamination Collar for RevoLock & RevoFit Lamination Kits with the Click Reel only.	
RP-201	Lamination Tube - 1.8m	Replacement Lamination Tube (1.8m) for all Revo Kits with the Click Reel and BOA® dials.	
RP-305	RevoLock® Tool Form Plug 5-Pack	The RevoLock Tool Form Plug 5-Pack is to ensure fabricators have extra plugs on hand.	
RP-300	Slider Part Kit	Replacement Slider and Lock Wire for dynamic AFOs, large gap prostheses or RevoLock applications.	
RP-309	Slider Lamination Kit	Lamination Kit for fabricating new dynamic AFOs, large gap prostheses or replacement for RevoLock Kits.	
BL-001	HC Lace - 25m Spool	25m spool of HC (High Capacity) Lace assures you will always have replacement lace on hand. Cost effective, sustainable and less waste. Used in the RevoSurface and RevoLock Kits with the Click Reel only.	
AC-100-05	Comfort Grip Ring - Black 5-Pack	Provide a soft-touch feel and 10% increased turning power to the existing Click Reel. Great for patients with dexterity issues.	
AC-101-05	Power Grip Ring - Black 5-Pack	Provide a soft-touch feel and 20% increased turning power to the existing Click Reel. Great for patients with dexterity and hand strength issues.	

SKU	Product	Description	Picture
RP-306	RevoLock® UE Fixed Insert	RevoLock Fixed Threaded Insert (M6) for upper extremity and pediatric liners. Compatible with RevoLock 4-Hole and RevoLock Align Kits. Best for patients who cannot operate the RevoLock Quick Connect Snap or for suction systems needing an airtight seal. This part is NOT releasable and does not connect to the RevoLock LE Quick Connect Snap.	
RP-307	RevoLock® LE Fixed Insert	RevoLock Fixed Threaded Insert (M10) for lower extremity liners only. Compatible with RevoLock 4-Hole and RevoLock Align Kits. Best for patients who cannot operate the RevoLock Quick Connect Snap or suction systems needing an airtight seal. This part is NOT releasable and does not connect to the RevoLock LE Quick Connect Snap.	
RP-303	RevoLock® LE Quick Connect Snap	Replacement RevoLock LE Quick Connect Snap connects the HC (High Capacity) Lace to the liner via the RevoLock LE Quick Connect Insert (RP-302).	
RP-302	RevoLock® LE Quick Connect Insert	Replacement RevoLock LE Quick Connect Insert for connection to lower extremity liners. Compatible with RevoLock LE Quick Connect Snap (RP-303) found in the RevoLock Align and RevoLock 4-Hole Kits. Often purchased for outfitting additional patient liners.	









QuickFit® Made with the BOA® Dial			
SKU	Product	Description	Picture
OK0180-130-05	QuickFit® Lacer, XS	<p>The extra small size QuickFit Lacer is designed to close a small region of an orthosis, a pediatric sized orthosis, or pediatric footwear up to size 35 EU.</p> <p>Comes pre assembled with a low-profile dial and 4 guides. QuickFit Rivets sold separately, but are required to install the QuickFit Lacer kits. Lacers are sold individually. For a pair of shoes, two kits are required.</p>	
OK0185-130-05	QuickFit® Lacer, XS 10-Pack	<p>The extra small size QuickFit Lacer is designed to close a small region of an orthosis, a pediatric sized orthosis, or pediatric footwear up to size 35 EU.</p> <p>Comes pre assembled with 10 extra small QuickFit Lacer kits and 100 QuickFit Rivets. For a pair of shoes, two kits are required.</p>	N/A
OK0200-130-05	QuickFit® Lacer, SM	<p>The small size QuickFit Lacer is designed to close a small region of an orthosis, a small sized orthosis, or footwear size 36 - 41 EU.</p> <p>Comes pre assembled with a low profile dial and 6 guides. QuickFit Rivets sold separately, but are required to install the QuickFit Lacer kits. Lacers are sold individually. For a pair of shoes, two kits are required.</p>	
OK0205-130-05	QuickFit® Lacer, SM 10-Pack	<p>The small size QuickFit Lacer is designed to close a small region of an orthosis, a small sized orthosis, or footwear size 36 - 41 EU.</p> <p>It comes with 10 Small QuickFit Lacer kits and 100 QuickFit Rivets. For a pair of shoes, two kits are required.</p>	N/A
OK0220-130-05	QuickFit® Lacer, MD	<p>The medium size QuickFit Lacer is designed to close a medium sized orthosis or footwear size 42 EU and larger.</p> <p>Comes pre assembled with a low profile dial and 8 guides. QuickFit Rivets sold separately, but are required to install the QuickFit Lacer kits. Lacers are sold individually. For a pair of shoes, two kits are required.</p>	

QuickFit® Made with the BOA® Dial			
SKU	Product	Description	Picture
OK0225-130-05	QuickFit® Lacer, MD 10-Pack	<p>The medium size QuickFit Lacer is designed to close a medium sized orthosis or footwear size 42 EU and larger.</p> <p>It comes with 10 Medium QuickFit Lacer kits and 100 QuickFit Rivets. For a pair of shoes, two kits are required.</p>	N/A
OK0240-230-05	QuickFit® Lacer, LG	<p>Designed to close or secure above the ankle shoes or orthoses, such as a Leather AFO's, plastic rigid AFO's, or Crow boots. Can also be used on footwear such as hiking boots.</p> <p>Comes pre assembled with a more powerful dial and 10 guides. Rivets sold separately.</p>	
OK0165-230	QuickFit® Releasing Lacer, LG	<p>Adds opening functionality to our QuickFit Lacer by adding releasing guides to one side so that the system can fully open like a door for easy donning.</p> <p>Comes with a more powerful reel to close larger devices. This kit is unassembled with 4 fixed guides and 3 releasing guides. Rivets sold separately.</p>	
TL-001	QuickFit® Tool	<p>The QuickFit Tool is customized for use with QuickFit Rivets and is the recommended installation tool. The QuickFit Tool properly secures rivets and provides either an 1/8" and 3/16" rivet hole.</p>	
SP1040	QuickFit® Rivets 100-Pack	<p>QuickFit Rivets come in packs of 100. You can supply your own rivets, or purchase our easy to use rivet pack to install QuickFit Lacers to any device or shoe.</p>	


QuickFit® Made with the BOA® Dial			
SKU	Product	Description	Picture
RP0200-130-05	QuickFit® XS/SM/MD Dial and Lace Replacement Kit	Replacement BOA® dial and 1.5m of Low-Mid Power Steel Lace for XS/SM/MD size QuickFit Lacers using BOA® dials.	
RP0200-230-05	QuickFit® LG Dial and Lace Replacement Kit	Replacement BOA® dial and 2m of compatible Low-Mid Power Steel Lace for LG size QuickFit Lacers using BOA® dials.	
LOK019	Low-Mid Power Steel Lace - 10m Spool	Bulk replacement Low-Mid Power Steel Lace (10m spool) compatible only with BOA® dials and QuickFit Lacer kits. NOT COMPATIBLE with Click® Reel.	N/A
OK1410-130-05	QuickFit® Strap 1.5"	QuickFit Straps are designed to quickly install on a wide variety of cuffs, orthoses, and prosthetic devices to create an improved, easy to use, adjustable strap. <ul style="list-style-type: none">• Prefabricated, durable, and installs in minutes• Don/doff via simple Velcro® closure• 100mm (4") of adjustable travel• Usable strap length: Max: 0.75m (30") / Min: 0.2m (9")• Cut to size• Width (38mm)	
OK1415-000-05	QuickFit® Buckle 1.5"	The QuickFit Buckle can be added onto the QuickFit Strap in a few seconds. The magnetic buckle allows the user to easily connect the strap - even with limited dexterity or in hard to reach areas. Its plastic on plastic interface eliminates wear issues of Velcro® closure. <ul style="list-style-type: none">• Easily adds onto QuickFit Strap in seconds• Adds 100mm (4") to strap usable length• 5mm (.2") profile• Width (38mm)	

Notes:


Replacement Parts using the BOA® dial			
SKU	Product	Description	Picture
PK0060-000-05	RevoFit® Re-Lacing Kit for BOA® dial	2m of HD (High Durability) Lace for RevoFit Lamination Kits Compatible only with RevoFit Kits with BOA® dial.	
RP0100-320-05	RevoFit® Dial and Lace Replacement Kit with BOA® dial	Replacement BOA® dial and 2m of HD (High Durability) Lace Compatible only with RevoFit Kits with BOA® dial.	
RP3000-220-05	RevoLock® Lanyard Dial and Lace Replacement Kit with BOA® dial	Replacement BOA® dial and 1.2m of lace Compatible only with RevoLock Kits with BOA® dial.	
RP3001-000-05	RevoLock® Lanyard Re-Lacing Kit for BOA® dial	Contains 1.2m of lace Compatible only with RevoLock Kits with BOA® dial.	

Replacement Parts using the BOA® dial			
SKU	Product	Description	Picture
RP3002-000-05	RevoLock® Lanyard - Lower-Ex Threaded Insert (10mm)	Replacement RevoLock Lower-Extremity Threaded Insert which attaches the lace to the liner via a threaded connection. Compatible only with RevoLock Kits with BOA® dial.	
RP3003-000-05	RevoLock® Lanyard - Upper-Ex Threaded Insert (6mm)	Replacement RevoLock Upper-Extremity Threaded Insert which attaches the lace to the liner via a threaded connection. Compatible only with RevoLock Kits with BOA® dial.	
LOK020	RevoLock® Soft Lace - 10m Spool	Bulk replacement lace (10m spool) Compatible only with RevoLock Kits with BOA® dial.	N/A

DIGITAL CATALOG FOUND HERE

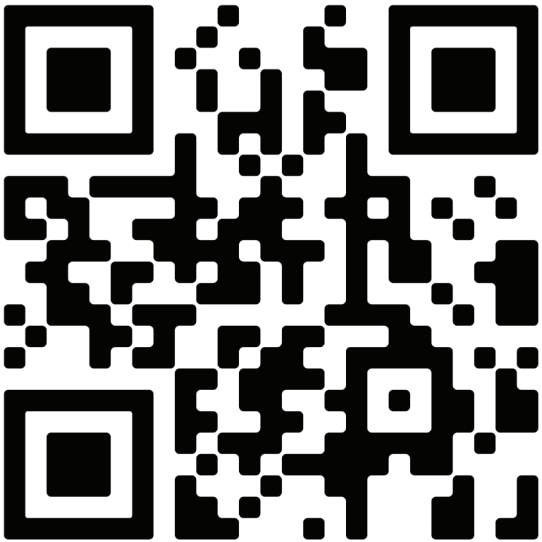

Find Click Medical® Adjustable Solutions Worldwide

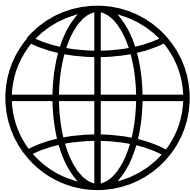

DISTRIBUTOR	COUNTRIES	ADDRESS	PHONE	WEB ADDRESS
NORTH AMERICA				
SPS	United States (includes Puerto Rico)	6530 Corporate Court Alpharetta, GA 30005 United States	Tel: +1 (800) 767-7776 ext.3	spsco.com
Cascade	United States (includes Puerto Rico) / Canada	2638 Aztec Drive Chico, CA 95928 United States	Tel: +1 (800) 888-0865	cascade-usa.com
OrtoPed	Canada	373 Rue McCaffrey Montréal, QC H4T 1Z7 Canada	Tel: +1 (514) 342-6161	https://www.ortoped.ca/
PEL Supply	United States	4666 Manufacturing Avenue Cleveland, OH 44135-2672 United States	Tel: 1-800-321-1264	Pelsupply.com
Ortiz International	Mexico	Federalismo Sur 836 Guadalajara, Jalisco, 44190 Mexico	Tel: +33 3811-2151 6 3811-3827	https://ortizinternacional.com.mx/
EUROPE				
Alfimed AS	Norway, Sweden, Denmark, Finland	Slemmestadveien 636 3470 Slemmestad, Norway	Tel: +47 31 29 68 86	www.alfimed.no
EMO	Spain, Portugal	Camino Pascualeta, 40. 46200 Paiporta, Spain	Tel: +34 963 132 051	emo.es/es
Frey Orthopädie-Bedarf AG	Switzerland	Panoramaweg 35 5504 Othmarsingen Switzerland	Tel: +41 62 887 45 00	freyortho.ch/en/
Groupe G2M	France	Chemin des Aubrières 49400 Saumur France	Tel: +(33) 2 41 50 50 91	groupe-g2m.com
ING Corporation	Czech Republic, Romania, Slovakia, Croatia, Latvia	Mánesova 1259, 739 11 Frýdlant nad Ostravicí Czech Republic	Tel: +(420) 558 663 217	ingcorporation.cz
Mediroyal	Norway, Sweden, Denmark, Finland, Iceland, Estonia	Box 7052 Staffans väg 6 B S-192 07 Sollentuna Stockholm, Sweden	Tel: +46 8 506 766 00	https://www.mediroyal.se/en

Metiz IMPEX	Russia	Kolpakova 24/A, Mytischí, Moscow region, 141008 Kolpakova 24/A Mytischí, Moscow region 141008 Russia	Tel: +7 (495) 215-53-82	https://metiz-ltd.com
BeMedico	Belgium, Netherlands, Luxembourg	Mutsaardstraat 47 9550 Herzele Belgium	Tel: +32 (0) 54 50 40 05	https://bemedico.be/
Ottobock	Germany, (worldwide)	Max-Näder-Straße 15 (or Max-Naeder-Strasse 15) D-37115 Duderstadt Germany	Tel: +49 5527 848-0	ottobock.com
Protheseus GmhB	Germany, Austria	Liebigstraße 13 91126 Schwabach Germany	Tel: +49 (0) 9122 88 707-0	https://www.protheseus.de/
Simex	Ukraine	Building 3, Letter “U”, str. Glubochitska 17 Kiev, Ukraine, 04050	Tel: +(38) 044 4861406	www.parasind.com
Steeper	UK, Ireland	Unit 3, Stourton Link Intermezzo Drive Leeds LS10 1BL UK	Tel: +44 (0) 7980190698	steepergroup.com
MIDDLE EAST				
Gilron	Israel	2 Haron Katzir Ave Tel-Hashomer, Ramat-Gan, Israel	Tel: +(972) 3-5409333	https://gilron.com/
Ortotek Ortopedi Ltd.	Turkey	Ziya Gökalp Cad.No:68/18 Kolej/Ankara – Turkey	Tel: +(90) 3124301830	www.ortotek.com
AUSTRALIA				
Massons	Australia, New Zealand, Singapore	Unit 15/111 Lewis Rd Knoxfield, Victoria, 3180 Australia	Tel: +61 3 9898 3440	massonshealthcare.com.au
ASIA				
Imasen Engineering Corporation	Japan	3-1-8 Techno Plaza Kakamigahara Gifu JAPAN 509-0109	Tel: +(81) 90-2348-9484	imasengiken.co.jp/
Teh Lin Prosthetic & Orthopaedic Inc.	China, Taiwan	No.7, Wuquan 7th Rd., Wugu Dist., New Taipei City 248 New Taipei City, Taiwan 248 Taiwan, China	Tel: +(886) 2-22991901	www.tehlin.com
SOUTH AMERICA				
Ethnos	Brazil	Rua 15 de Novembro, 469, Vila São Luiz, Duque de Caxias Rio de Janeiro, Brazil	Tel: +55 21 2775 4032	https://ethnos.com.br/

U.S. Insurance Coding

RevoSurface®

New Fabrication of Orthoses:

L2330 - Lacer molded to patient model for custom fabrication of an orthosis.

Repair of an existing Prosthesis:

L7510 for cost of materials and L7520 for repair labor component.

Repair of an existing Orthosis:

L4210 for cost of materials and L4205 for repair labor component.

RevoLock® 4-Hole & Align

L5671 - Lower Extremity and Replacement

L6698 - Upper Extremity

RevoFit®

Within the U.S.A. there is not a specific code for RevoFit®, however we recommend using our resources for reimbursement success on our website:

clickmedical.co/revofit - scroll down to Insurance Coding.

If using RevoFit® in a PDE orthosis use L2330 - Addition to lower extremity, a lacer molded to patient model, for custom fabricated orthosis only.

QuickFit®

Lacers:

- **Creating a lacer:** L2320 or L2330
- **Creating a shoe:** L5999 > “same or similar as L3580”

Straps and Buckles:

- **Repair of an existing Prosthesis:** L7510 for cost of materials and L7520 for repair labor component.
- **Repair of an existing Orthosis:** L4210 for cost of materials and L4205 for repair labor component.



Adam Griggel
1983-2022

Adam was a true advocate for himself and for others in the amputee community. He was a dear friend of Click Medical and he will be missed.

Adam Griggel, RevoFit user and son, Lincoln.

Questions?

We are here for you!

Technical Support

M - F, 9am - 5pm MST
+1 (970) 670-7012

Online Support Form:

clickmedical.co/contact-us

Click Medical

1205 Hilltop Parkway
W101
Steamboat Springs, CO
80487



You build it. They adjust it.

Adjustability will revolutionize your practice.

clickmedical.co