

QuickFit™ Strap

by CLICK

		WARRANTY US customers contact Click Medical directly: Help@ClickMedical.co Help Line: +1.970.670.7012 International customers, please contact your local distributor					
<table border="1"><tr><td colspan="2">MD</td></tr><tr><td>EC</td><td>REP</td></tr></table>		MD		EC	REP	MDSS GmbH. Schiffgraben 41 30175 Hannover Germany	
MD							
EC	REP						
<table border="1"><tr><td>CH</td><td>REP</td></tr></table> MDSS CH GmbH. Laurenzenvorstadt 61 5000 Aarau Switzerland		CH	REP	<table border="1"><tr><td>EU</td><td>REP</td></tr></table> MDSS-UK RP LIMITED 6 Wilmslow Road, Rusholme M14 5TP Manchester United Kingdom		EU	REP
CH	REP						
EU	REP						
		Click Medical, LLC 1205 Hilltop Parkway, W101 Steamboat Springs, CO, 80487, USA • 1-970-670-7012					

Revised 2022

QuickFit™ Strap Fitting Instructions

Kit Contents


QuickFit™ strap x 1

38 mm/1.5" - OK1410-130-05




Adhesive Velcro® tab x 2

1 Install Velcro® at edges of device.

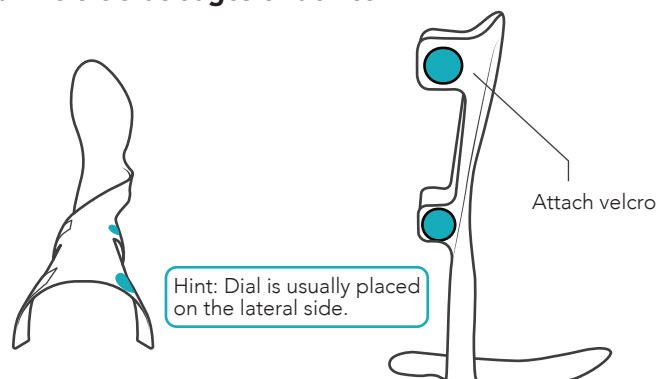


2 Check strap position.


Important: Add optional QuickFit™ Buckle if desired before continuing to step 3 (see other side for instructions)

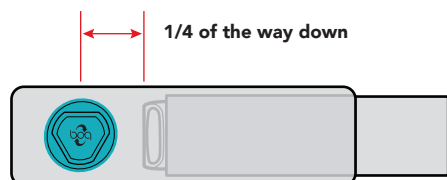


3 Position tunnel opening at edge of desired closure.

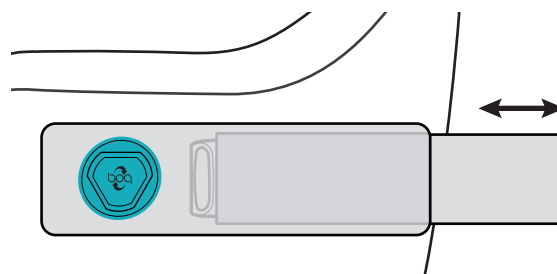

4 Wrap strap around body to determine strap length.

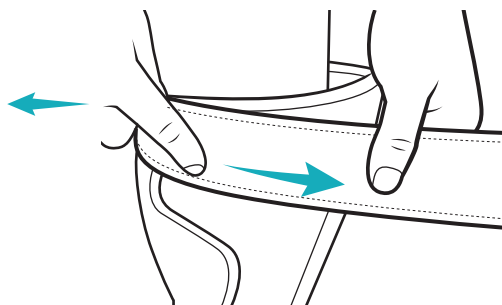


5 Cut tail of strap to size at dial.

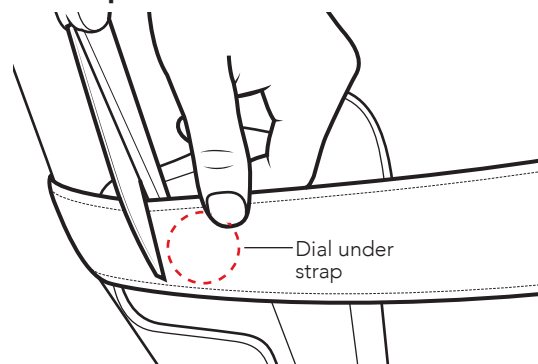

6 Push to engage dial. Twist to tighten.



7 Pull up on dial to release tension. Lift tab to release.

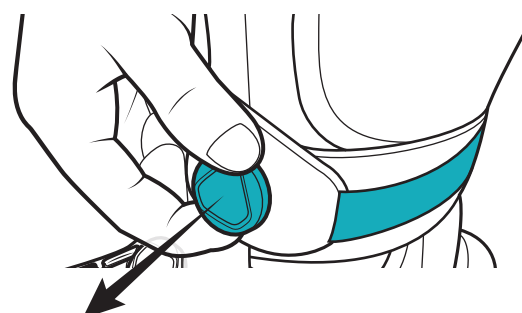

QuickFit™ Buckle Add-On Instructions


by CLICK

1" / 25mm OK1405-000-05

1.5" / 38mm OK1415-000-05


2" / 50mm OK1425-000-05

- 1 Install QuickFit™ buckle to device with included Velcro® tabs.

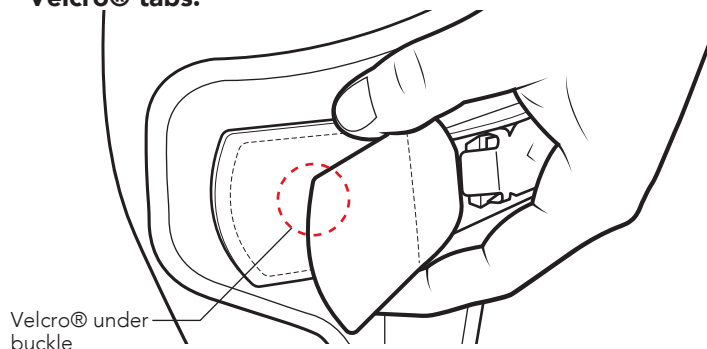


- 2 Attach QuickFit™ strap.


- a. Insert tab through back of first bar.

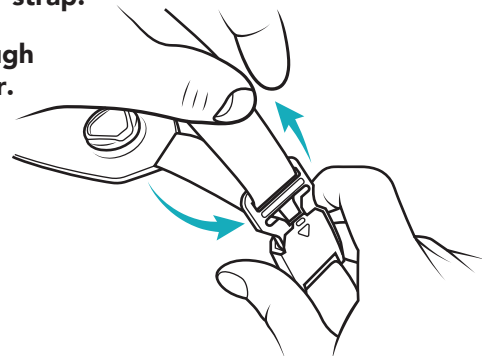


- b. Loop tab back through adjustment bar.

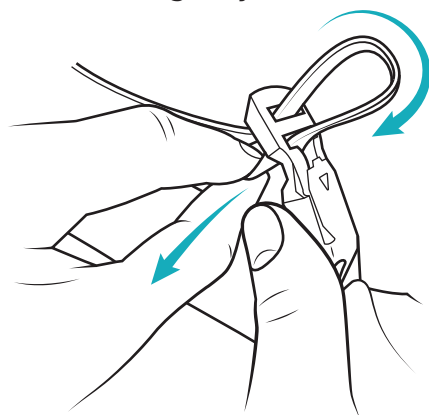

- c. Secure tab to back of tunnel.

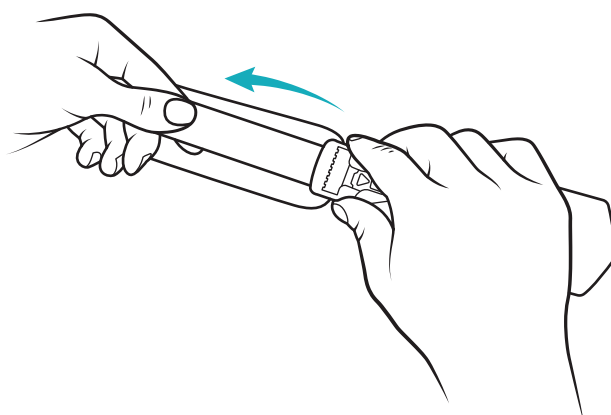


- 3 Cut to size, attach tail of strap to buckle.

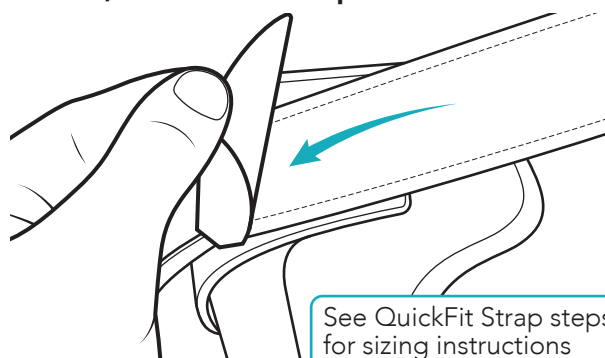


See QuickFit Strap steps 2-4 for sizing instructions

- 4 a. Close strap at self-securing buckle.
b. Push to engage. Twist clockwise to tighten.

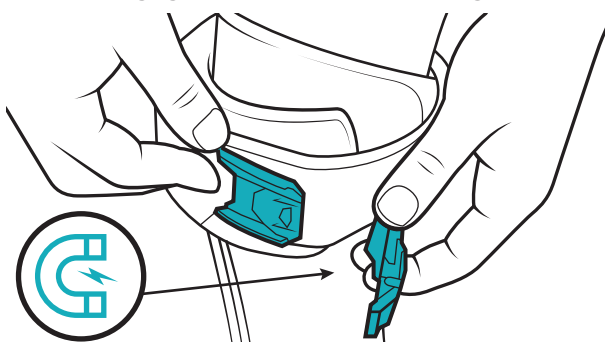

To Release Strap

- 1 Pull up dial for quick release.

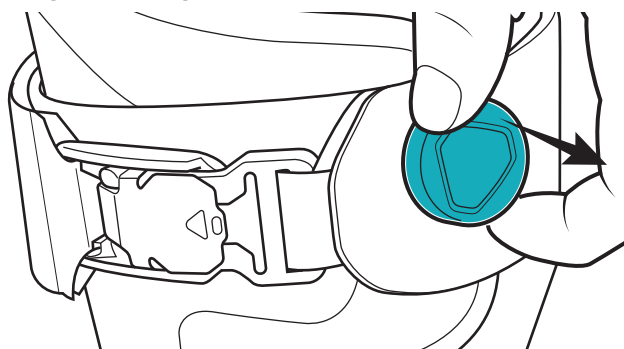

- 2 Pull tab to open.

